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Recommendations for Use and Safe Operation in Dental Practices

1. General

To prevent harm to the residual tooth structure and the tissues and teeth contiguous to the site, provide for sufficient cooling, technically, clinically and hygienically optimum instruments and materials as well as optimum instrument speed and only exert the necessary amount of pressure. The service-life of diamond instruments depends on compliance with the following recommendations for use and safe operation.

2. Recommended hygiene procedures

The diamond instruments are selected (shape, size, type) according to the type of preparation to be carried out.

The ergonomic principles must be taken into account during the working procedures.

The handpieces (high-speed handpieces, contra-angles) for diamond instruments must be in perfect working conditions.

The instruments must be fully inserted and gripped in the high-speed handpiece or contra-angle with utmost care.

Select the correct speed to ensure optimum performance and service-life. Operate within the recommended speed ranges.

The hints covering maximum permissible speeds must be heeded.

Before placing the diamond in contact with the site being prepared, check that the bur is rotating at the optimum speed and that the cooling system is functioning properly. Once the preparation has been completed, remove the rotary instrument from the site and allow it to come to a standstill.

Prepare carefully and without exerting pressure. Select the pressure (0.3 – 2N) to ensure that the speed is not reduced noticeably. The pressure exerted on the rotary instrument is a decisive factor for the working procedure and the successful outcome of the preparation. Under no circumstances should the instrument be allowed to jam.

To prevent harm to the tooth structure, pulp and adjacent fillings, cool by spraying with sufficient amounts of water. We recommend high-speed handpieces with three jets which spray the coolant along the entire length of the instrument's cutting surface. Preparing a tooth without using sufficient coolant – minimum: 50ml/min. and incorrect application of the coolant (spray diverted, inadequate ejection) may adversely affect the final result.

Diamond instruments of excessive length (>21mm) and diamond instruments with a head exceeding 2mm in diameter require additional cooling.

Only use impeccable diamond instruments. Instruments which are blunt, damaged, bent or no longer concentric, must be segregated immediately and disposed of carefully.

Whenever possible, the diamonds should be stored in their original packaging to enable them to be identified and traced.

Recommended Hygiene Procedures for Dental Practices

1. General

Protection against infection is especially important to patients and surgery staff. Disinfection and/or sterilization of the equipment and instruments used

are an absolutely essential part of a safe preparation technique.

- Diamond instruments must be sterilized prior to use.
- The following hygiene procedures are recommended before and after use:

2. Recommended hygiene procedures

Cleaning, disinfecting, sterilizing and storing diamond instruments:

- After use, instruments must be immersed immediately in a tank containing suitable cleaning agent/disinfectant. Use a disinfectant/cleaning agent containing a corrosion inhibitor. The cleaning agent/disinfectant manufacturer's instructions must be heeded.
- Strongly acidic and alkaline disinfectants may adversely affect the instruments. Strong chemicals attack the instruments.
- If necessary, clean by mechanical means in advance (brush the bur blades with a brush with nylon or brass bristles).
- Check for any remains and, if necessary, clean again.
- The instruments may be cleaned ultrasonically. The use of bur stands or holders is advisable as they prevent the instruments vibrating against each other and becoming damaged.
- Sterilize the instruments in a chemiclave, autoclave or in hot air.

Hints:

- Avoid temperatures exceeding 200°C.
- Sterilizers must be used in accordance with the manufacturer's instructions.
- The instruments should then be stored in alcohol or ultra-violet surroundings until further use.
- For long-term storage, keep the instruments in stands or containers in a dry place, protected against dust and acidic vapours.

Please note:

To prevent spreading infectious germs, instruments contaminated with blood or saliva should be disposed of carefully

Recommended Speeds

Head diameter in 1/10 mm	Speed range (r.p.m.)
008 – 010	75.000 – 150.000
012 – 014	60.000 – 110.000
016 – 018	45.000 – 88.000
021 – 023	40.000 – 75.000
025 – 027	30.000 – 65.000
029 – 031	25.000 – 56.000
033 – 040	22.000 – 45.000
042 – 050	20.000 – 37.000
055 – 060	17.000 – 32.000
065 – 080	13.000 – 26.000
085 – 100	12.000 – 24.000
120 – 140	8.000 – 15.000
160 – 180	6.000 – 12.000
200 – 220	5.000 – 11.000

Marking

The **LOT** number is shown on the package

Date of issue:

06-2017

Only supplied to dentists and dental laboratories or upon their instructions.

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